My WHITE

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My White - Instructions of use

My White Evolution is an in-office professional system for bleach intrinsic tooth discoloration by dental professionals only. It can be used in vital and non vital teeth.

COMPOSITION

Evolution 1 contains:

- 1 predosed vial with 5ml of hydrogen peroxide 38%
- 1 container with 1,6g (powder)
- 1 spatula.
- 1 patient card.

Evolution 4 contains:

- 4 predosed vials with 5ml of hydrogen peroxide 38%
- 4 containers with 1,6g (powder)
- 1 syringe of 3,5g light cure gingival protection My White Mask.
- 4 metal tips.
- 4 spatulas.
- 4 patient cards.

INDICATIONS

It is used to bleach a single tooth or several teeth with discoloration or devitalized. The discoloration may be caused by: medicines, fluorosis, food stains and aging. It is recommended to complete the bleaching treatment at least two weeks before placement of restorations. Existing restorations, which the product do not effect, might not match the lighter color of the teeth after the treatment and might need to be replaced.

CONTRAINDICATIONS

The product is not recommended:

- for pregnant or breastfeeding mothers.
- patients under 18 years old.
- patient with gingival disease, allergic to any of the product components, with exposed root, with primary and secondary untreated caries, with crowns and bridges with marginal cracks.

SIDE EFFECTS

There might be a temporary sensitivity of the oral mucosa or the teeth; these symptoms usually disappear when the treatment is finished or interrupted in a short time.

WARNINGS

The hydrogen peroxide in the kit is corrosive. ALWAYS WEAR PROTECTIVE EYEWEAR AND GLOVES. Avoid the contact of the gel with the skin and oral mucosa; in case of contact with skin or oral mucosa, rinse plenty with water. Consult a doctor if the irritation persists. In case of contact the gel with eyes, rinse plenty with water for at least 15 minutes and consult immediately the doctor. In case of swallowing the gel, drinks plenty of water and consult the doctor. In case of inhaling the powder, breath at open and well ventilated area. If breathing problem persists, consult a doctor. Make the complete treatment at maximum 2 times in an year per patient. The doctor must check the integrity of the packaging before using the product. Dispose the packaging according to the local and governmental authorities regulations. Do not use the product with a different purpose from those which it was projected. Store the product at a temperature between 5 °C and 25 °C in a cool, dry, and ventilated place. Keep away from heat and light. Keep out of children's reach.

NOTES

To be used exclusively in oral treatment as described in the illustrative brochure. The product must be used according with the specific instructions of use. My Med kft does not accept any responsibility for damages from misacting the instructions of use or for rather uses of those foreseen by the application of the product. The user is responsible for experimenting the material which is not explicitly described in the instructions of use. My Med kft informs as being a product solely for professional purposes, shall not be liable for any physical consequence went to children and adults related to the careless of the product.

APPLICATION

Before to start the bleaching procedure, the patient should be informed about the complete procedure as well sign all formal documentation. A patient history should be made by the dentist deciding the treatment to be done. Make sure that the teeth to be treated are deeply clean and proceed with initial shade evaluation. Use gloves,

My White - Instructions of use

masks and protective goggles before the gel preparation.

Gel preparation

- a. Open My White Base container
- b. Cut the tip part of My White Activator predosed vial and pour into My White Base container
- c. Mix with the spatula until a soft viscosity gel is obtained. The gel will have a bright pink color.

Patient Preparation and gel application

1. Lift the lip with a mouth opener and cotton rolls in order to isolate the area.
2. Remove the cap at the My White Mask syringe and insert the metal tip with a clockwise rotation until it blocks.
3. Extrude the protective gel around the gingival edges of the superior jaw (area which will be treated) and light cure. Repeat the procedure in case that any edge were not well covered and light cure again. Repeat the gingival protection operation at the inferior jaw controlling that all gingival edges are covered and light cure.
4. Using the spatula, take the prepared bleaching gel from the container and apply it all over the teeth which will be treated. It should have at least an uniform gel layer of 1mm thick.
5. Activate the bleaching process (equipment necessary but not supplied with the kit). My White Evolution gel can be activate with several type of curing light (example: composite led light). The important is to respect the minimum power required: 1000 mW/cm². In case of use a bleaching light (lamp which irradiate the both jaws simultaneously) the procedure time would be much shorter.

Option A

ACTIVATION TIME WITH CURING LIGHT

My White gel activation time depends the light power output. It might vary from approximately 5 seconds to 20 seconds per tooth. The chromatic change indicates that the gel is activated. Exposing the gel for over irradiation do not increase the bleaching result and might cause patient discomfort. Start irradiating one tooth and when notice the chromatic change of the gel, goes to the next tooth. After irradiating all teeth (1cycle), remove the product using a gauze and/or with the suction tip. Repeat the gel application and irradiation procedure for 3 more cycles. Start the first cycle at the superior jaw and then at the inferior jaw alternatively. It is a four cycle protocol with an optional two extra circles protocol in complete safety.

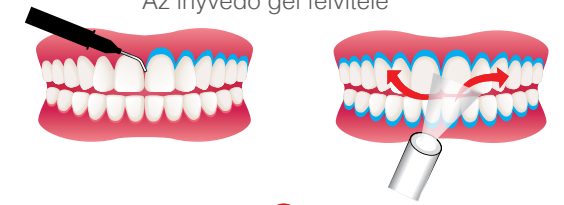
Option B

ACTIVATION TIME WITH BLEACHING LIGHT


My White gel activation time using a bleaching light (minimum power output: 450mW/cm²) would be maximum of 3 minutes each cycle. Remember that the chromatic change indicates that the gel is activated. Exposing the gel for over irradiation do not increase the bleaching result and might cause patient discomfort. Program the bleaching lamp for 3 minutes and starts the irradiating both jaws. When the cycle is finished, remove the product using a gauze and/or with the suction tip. Repeat the gel application and irradiation for 3 more times. An optional 2 extra cycles protocol in complete safety is at the discrepancy of the operator.

6. Remove the dam used to protect the gum and rinse with plenty of water. Do not forget to inform the patient relating to the precautions post treatment.

3
Protection gingival gel application
Applicazione della diga liquida
Az invédő gél felvitele

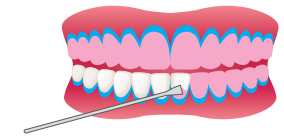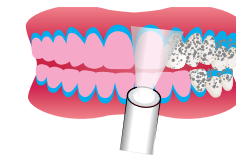
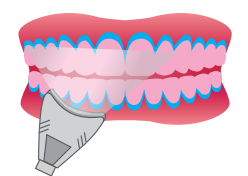
4
Bleaching gel application
Applicazione del gel sbiancante
A fehéritő gél felvitele

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