

Distributed by:
Distribuido por: Vertrieb durch:
Distribué par: Distribuito da:

HENRY SCHEIN INC.
Melville, NY 11747 USA

EC REP

Henry Schein U.K. Holdings Ltd.
Gillingham ME8 0SB U.K.

www.henryscheinbrand.com

Please note:

The mini file holder is not supplied
in a sterile state!

MADE IN ISRAEL

Fabricado en Israel/Fabriqué en Israel/Hergestellt in Israel/Prodotto in Israel

HENRY SCHEIN®

900-3643

MINI FILE HOLDER

*INSTRUCTIONS
FOR USE*

Congratulations!

You have purchased the
most advanced file holder
available for use with your
Apex Locator.


** See other side for detailed photo*



INSTRUCTIONS:

- Insert the mini file holder (2) into the socket at the tip of the cable (1).

Please note: the file holder should be inserted into the cable in one direction only - the slit opening for the file should be facing outwards, and not inside the socket.

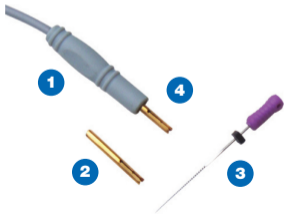

- File sizes thicker than 0.2, should be slipped into the first notch.

Inserting a large file into the second groove may widen the file holder and thin files may slip out. Thin files may be inserted into the second notch.

- Insert your file (3) either NiTi or regular) into the slit opening at the tip of the mini-file holder (2) until it reaches the groove (4).

The file is now secure in the holder.

- The mini file holder should appropriately sterilized after each use, and between patients.

Please note: The mini file holder is compatible with most endo handpieces